
















TROUBLESHOOTING GUIDE



● STEADY ○ FLASHING ○ OFF

LIGHTS		STATUS
 STELLAR Hybrid FiC	 CORE Hybrid FoC	○ FAULT FiC initialising
 STELLAR Hybrid FiC	 CORE Hybrid FoC	○ FAULT FiC set
 STELLAR Hybrid FiC	 CORE Hybrid FoC	○ FAULT FoC initialising
 STELLAR Hybrid FiC	 CORE Hybrid FoC	○ FAULT FoC set
 STELLAR Hybrid FiC	 CORE Hybrid FoC	● FAULT FiC failure, low pressure. Check connections, and cells for leaks. If not resolved then return for service.
 STELLAR Hybrid FiC	 CORE Hybrid FoC	● FAULT FoC failure, low pressure. Check connections, and cells for leaks. If not resolved then return for service.
 STELLAR Hybrid FiC	 CORE Hybrid FoC	● FAULT Compressor or system failure. Return for service.
 POWER FAILURE		Power Failure. Check plugged in and power available.