

## Quick User Guide for Core and Stellar Hybrid Mattress Systems

This is a quick user guide for IQ Medical's Core and Stellar Hybrid Mattress Systems.  
Also applicable for Core and Stellar Bariatric.

1



Place Hybrid Mattress on bed frame.

2



Run the mains cable through the cable management conduit, if applicable and plug into mains socket. Connect mains cable to pump.

3



Connect the air tube to pump via connector (an automatic click will signify a secure connection).

4



Switch the pump on via the POWER Button on Control Panel.

5



POWER LED will show green.  
If using CORE Hybrid, ensure CORE Hybrid FoC LED is flashing.  
If using STELLAR Hybrid, ensure STELLAR Hybrid FiC LED is flashing.

6



When correct pressure is reached, the SERVICE LOCK (either STELLAR or CORE) LED will stop flashing and remain constant. This signifies mattress has reached required pressure.